


T2 Alpha Tibia Nailing System

SPI Modular Handle assembly/disassembly

The T2 Alpha SPI Modular Handle comes in two pieces: a magnetically activated trigger and a handle.

Assemble or disassemble the SPI Modular Handle by inserting or removing the trigger respectively.

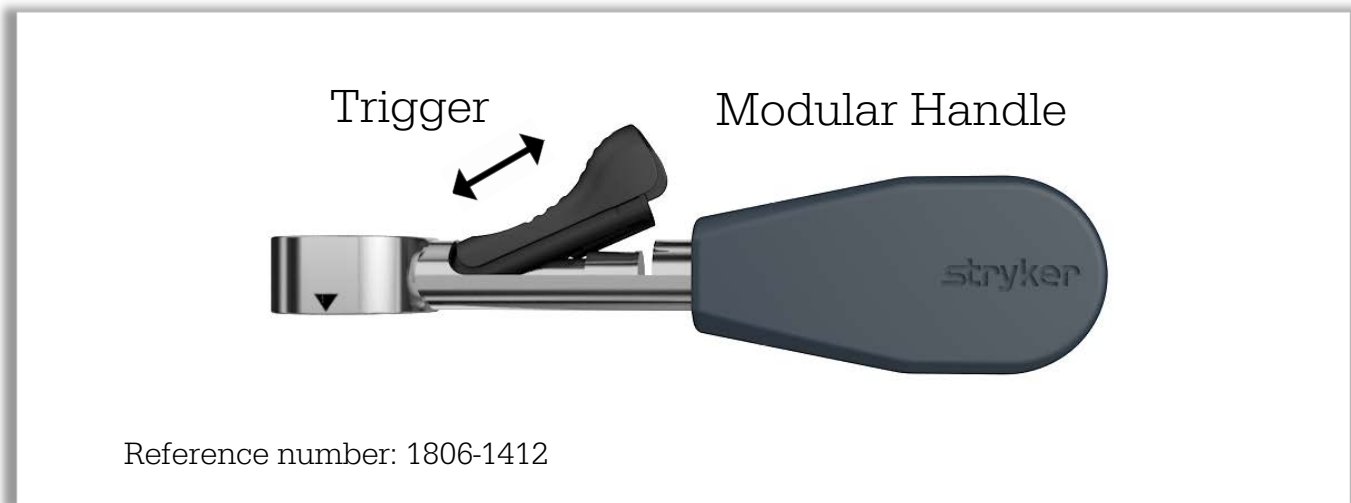

This poster displays selected key points and is not a substitute for the full Operative Technique manual.

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for Cleaning and Sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

The products listed above are CE marked.


Content ID: T2-ADI-2, 03-2020
Copyright © 2020 Stryker

Stryker Australia
8 Herbert Street,
St Leonards NSW 2065
T: 61 2 9467 1000 F: 61 2 9467 1010

Stryker New Zealand
515 Mt Wellington Highway,
Mt Wellington Auckland
T: 64 9 573 1890 F: 64 9 573 1891

T2 Alpha Tibia Nailing System

SPI Modular Handle tray location

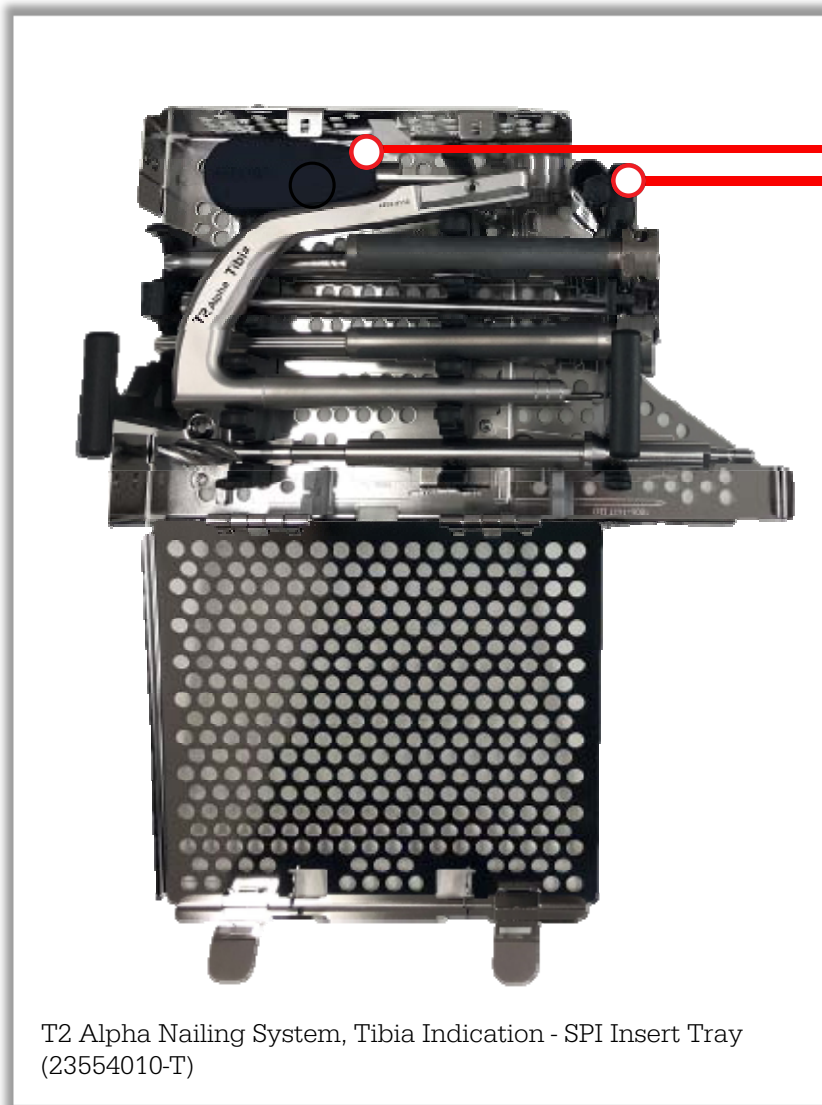

Modular Handle

Trigger

T2 Alpha Nailing System, Tibia Indication - SPI Insert Tray
(23554010-T)



This poster displays selected key points and is not a substitute for the full Operative Technique manual.

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for Cleaning and Sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its affiliates own, use, or have applied for the following trademarks or service marks: T2 Alpha, Stryker. All other trademarks are trademarks of their respective owners or holders.

Content ID: T2-ADI-2, 03-2020
Copyright © 2020 Stryker

Stryker Australia
8 Herbert Street,
St Leonards NSW 2065
T: 61 2 9467 1000 F: 61 2 9467 1010

Stryker New Zealand
515 Mt Wellington Highway,
Mt Wellington Auckland
T: 64 9 573 1890 F: 64 9 573 1891